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EN

Wrist braces support injured tissue structures (joint, ligaments, capsules, cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the joint. The thumb cut-out prevents the support from shifting. The aluminium splint inlay is formed in a way that the hand is hold in a relaxed, easy position. Additionally the splint offers increased protection against wrong movements.

Instructions for use:

Before applying bend the splints in desired shape. Push the hand into the support, fasten the wrist support Velcro.

Indications:

Pre-operative, post-operative, post-traumatic conditions, carpal tunnel syndrome (CTS), tenosynovitis, peripheral nerve paralysis.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

Composition:

83 % polyester, 17 % microfibre

Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

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